



## V.A.C.ULTA™ 4-Therapy System User Manual Addendum

This addendum is a supplement to the KCI V.A.C.ULTA™ 4-Therapy System User Manual, KCI p/n 416700. Each heading in this addendum corresponds to a heading within the main V.A.C.ULTA™ 4-Therapy System User Manual. The information under each of these headings is meant to supplement the information within those sections of the manual. The V.A.C.ULTA™ 4-Therapy System User Manual is also available on [www.veraflo.com](http://www.veraflo.com).

### Important Information about this Addendum

The V.A.C.ULTA™ 4-Therapy Plus upgrade adds the following new features:


- The ability to postpone the instillation phase of V.A.C. VERAFLOR™ Therapy for up to 60 hours and apply instead the V.A.C.® Phase of V.A.C. VERAFLOR™ Therapy. (refer to page 2 of this addendum).
- The ability to delay the Therapy Inactive Alarm when changing the dressing. This extension is only available and active when the dressing soak feature is selected. (refer to page 7 of this addendum).
- Several of the commonly asserted Alarms and Alerts will now present with animated help for resolving the condition of the alarm / alert. The animations depict the corrections provided in text in the user manual and provide ways to address the most frequent causes of alert/alarm condition. The animations are featured on the following alarms / alerts:
  - Blockage, Blockage (Therapy Interrupted)
  - Canister Not Engaged
  - Leak, Leak (Therapy Interrupted)
  - V.A.C. VERAFLOR™ Instill Tube Blockage (Therapy Interrupted)

Refer to the V.A.C.ULTA™ 4-Therapy User Manual for information on all Alerts and Alarms.

- An informational animation is also provided during initial draw down of the SEAL CHECK™ Leak Detector assessment. The steps depicted in the animation are to help correct leaks identified during the SEAL CHECK™ Leak Detector as outlined in the V.A.C.ULTA™ 4-Therapy User Manual.

## V.A.C. VERAFLOR™ Therapy Paused / Postpone

When **Pause/ Postpone** is selected from the **Home** screen (page 50 of the V.A.C.ULTA™ 4-Therapy User Manual) there are 2 options.

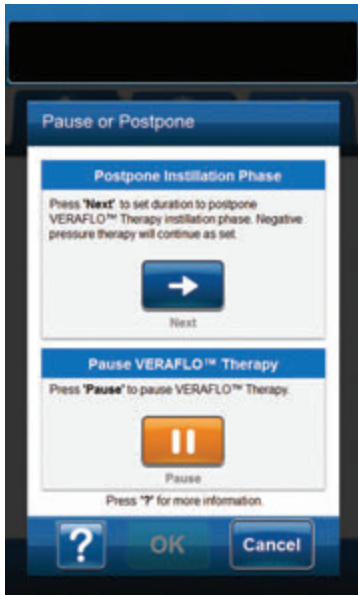

### To pause V.A.C. VERAFLOR™ Therapy:



Select **Pause** to continue to the **V.A.C. VERAFLOR™ Therapy™ Pause** screen (page 3 of this addendum).

### To postpone instillation phase of V.A.C. VERAFLOR™ Therapy:



Select **Next** to continue to the **Confirm Postponement** screen (page 5 of this addendum).

**OR**



Select **Cancel** to return to the **Home** screen for **V.A.C. VERAFLOR™ Therapy** without postponing therapy.

V.A.C. VERAFL0™ Therapy Pause



Select **OK** to pause **V.A.C. VERAFL0™ Therapy** and return to the **Home** screen.

*Therapy can be paused for up to 15 minutes. This may exceed selected soak time. Consider this pause time and the selected soak time before pausing therapy.*

**OR**


Select **Cancel** to return to the **Pause / Postpone** selection screen for **V.A.C. VERAFL0™ Therapy** without pausing therapy (page 2 of this addendum).



V.A.C. VERAFL0™ Therapy Pause - Resume Therapy



1. To resume V.A.C. VERAFL0™ Therapy, select **Resume** from the **Home** screen to continue to the **Resume VERAFL0™ Therapy** screen.

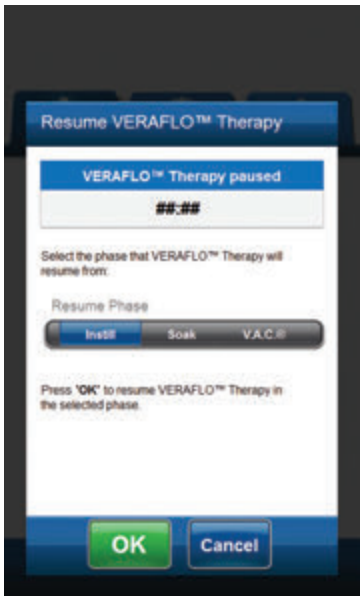


2. Select the phase that V.A.C. VERAFL0™ Therapy will resume from (**Instill**, **Soak**, **V.A.C.®**).




3. Select **OK** to resume therapy and return to the **Home** screen.

**OR**

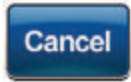

Select **Cancel** to return to the **Home** screen for **V.A.C. VERAFL0™ Therapy** with therapy paused.

## Confirm Postponement

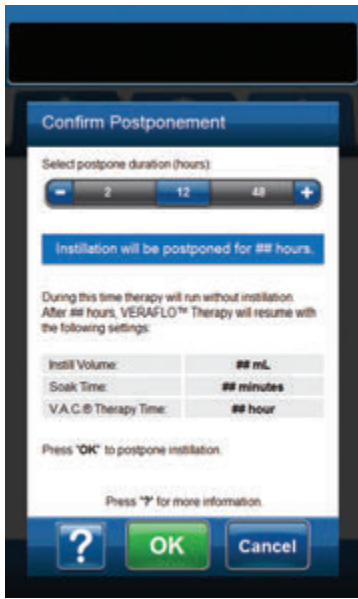

1. Use + / - to select the amount of time to postpone the instillation phase of V.A.C. VERAFLOR™ Therapy (2 hr - 60 hrs).



2. Select **OK** to continue to the *Home* screen with instillation paused.



***This feature allows the clinician to postpone the instillation phase of VERAFLOR™ Therapy and continue to provide V.A.C.® Therapy as previously confirmed.***

**OR**





Select **Cancel** to return to the *Pause/Postpone* selection screen for **V.A.C. VERAFLOR™ Therapy** without pausing therapy (page 2 of this addendum).



***Therapy will automatically resume once selected duration has elapsed.***

# Instillation Postponed - Resume Therapy

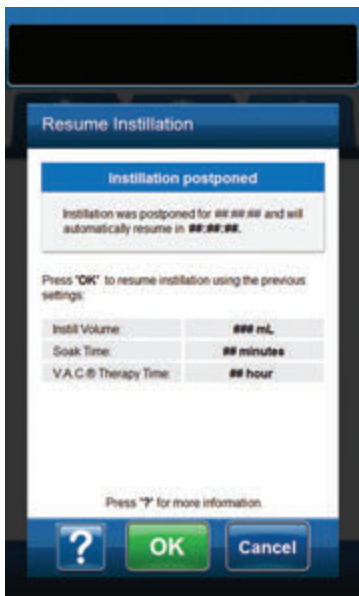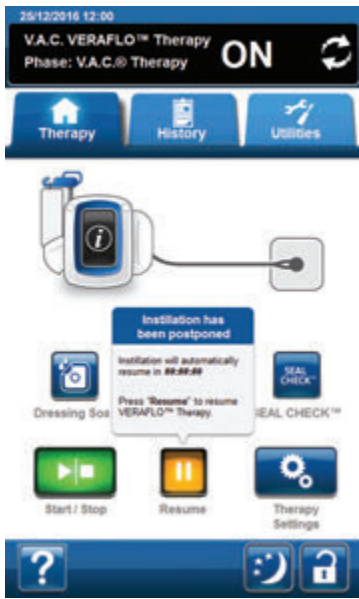

1. Select **Resume** to continue to the **Resume Instillation** screen to return to previously confirmed **V.A.C. VERAFLU™ Therapy** settings.



2. Select **Start / Stop** to stop therapy.



3. Select **OK** to resume therapy and return to the **Home** screen.



*Start phase will be instillation.*

**OR**




Select **Cancel** to return to the **Home** screen for **V.A.C. VERAFLU™ Therapy** with therapy postponed.

## Dressing Soak with Therapy Inactive Alarm Delay

Use to soak the dressing with solution in preparation for a dressing change and temporarily delay Therapy Inactive Alarm.



**If the Dressing Soak tool is available for selection, the Dressing Soak icon on the Home screen or Utilities Tab (V.A.C. VERAFLOR<sup>TM</sup> Therapy and V.A.C.<sup>®</sup> Therapy only) will be blue. Refer to page 75 - 78 in the V.A.C.ULTA<sup>TM</sup> 4-Therapy User Manual for details on Dressing soak while using V.A.C. VERAFLOR<sup>TM</sup> Therapy and page 107 in the V.A.C.ULTA<sup>TM</sup> 4-Therapy User Manual for details on Dressing soak while using V.A.C.<sup>®</sup> Therapy.**

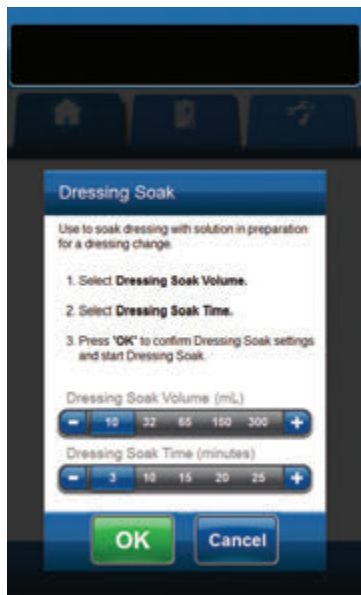

### Dressing Soak selected while therapy is idle:

1. Ensure that both the V.A.C.<sup>®</sup> canister tubing and instillation line are properly connected.
2. Ensure that all four tubing clamps are open.
3. Ensure that the V.A.C. VERALINK<sup>TM</sup> Cassette is properly installed (page 28 of the V.A.C.ULTA<sup>TM</sup> 4-Therapy User Manual).
4. Ensure that the canister has adequate capacity remaining for the dressing change.



5. Select **Dressing Soak** from the **Home** screen or **Utilities Tab** to continue to the **Dressing Soak** screen.
6. Select the target **Dressing Soak Volume (mL)**.




**Depending on which phase of V.A.C. VERAFLOR<sup>TM</sup> Therapy dressing soak is initiated the dressing soak volume may not be a selectable option.**

7. Select the target **Dressing Soak Time (minutes)**.



8. Select **OK** to confirm settings and return to the **Home** screen or **Utilities Tab**.
9. The V.A.C.ULTA<sup>TM</sup> Therapy Unit will complete the Instill, Soak, and fluid removal phases. Therapy phase will be displayed in the status bar (page 50 of the V.A.C.ULTA<sup>TM</sup> 4-Therapy User Manual) at the top of the screen. The current therapy status will also appear under the icon of the therapy unit along with time or fluid amount (during the **Instill** phase) remaining.

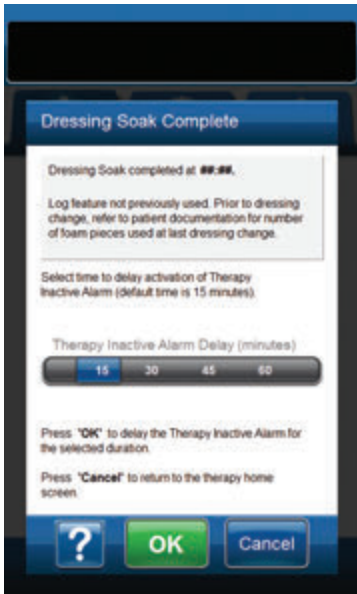

- Once the Dressing Soak fluid removal phase is complete, select the length of time to delay the Therapy Inactive Alarm (in minutes) and select **OK** to continue to the **Therapy Inactive Alarm Delayed** screen. The dressing can be removed.




- Select **Cancel** to return to the **Home** screen with therapy OFF.



**Refer to the appropriate dressing Instructions for Use for safety information and procedures to change the dressing.**

**When task is complete and delay is no longer needed:**

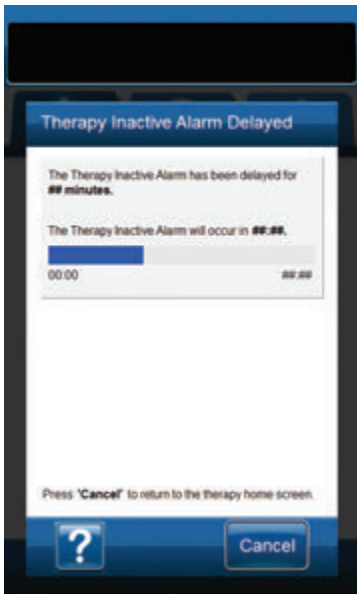

- Select **Cancel** to return to the **Home** screen (page 50 of the V.A.C.ULTA™ 4-Therapy User Manual) with therapy OFF.



**Refer to the appropriate dressing Instructions for Use for safety information and procedures to change the dressing.**











**Manufactured for:**

KCI USA, Inc.  
12930 IH 10 West  
San Antonio, TX 78249  
1-800-275-4524  
[www.acity.com](http://www.acity.com)  
[www.veraflo.com](http://www.veraflo.com)  
[www.vaculta.com](http://www.vaculta.com)

Trademarks designated herein are proprietary to KCI Licensing, Inc.  
Copyright 2019 KCI Licensing, Inc. All Rights Reserved. 420801 Rev A 12/2019